ALITA

# ALITA

## Support Monitor

[www.alita3x.com](http://www.alita3x.com)  
[support@alita3x.com](mailto:support@alita3x.com)


# PART I

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[www.alita3x.com](http://www.alita3x.com)  
[support@alita3x.com](mailto:support@alita3x.com)

# INTRODUCTION DE SUPPORT MONITOR

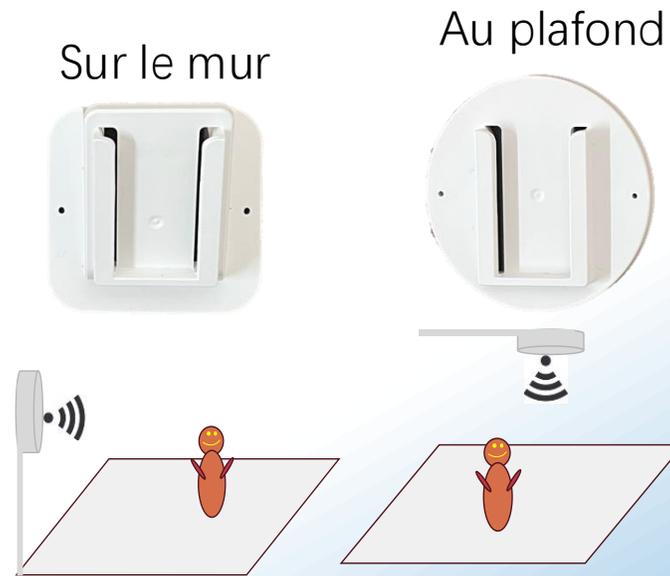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

67 27

1

The card displays patient information for Ishii Haruko in room 208A. It includes a call icon, a sleep icon with the number 1, and vital signs: 67 (heart rate) and 27 (respiration rate). Arrows point from the call icon to the 3D room view and from the sleep icon to the 360-degree view.

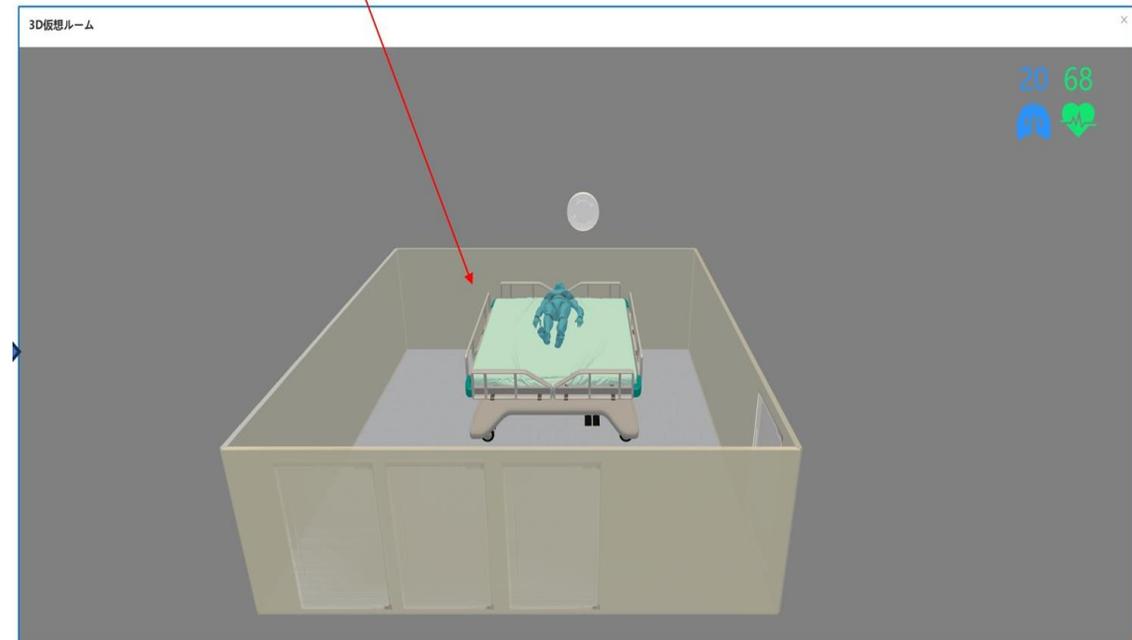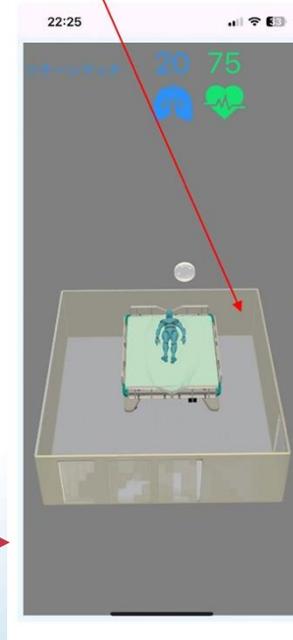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The screenshot shows a web browser at care.alita3x.com displaying a grid of resident health reports. A red circle highlights a document icon in the top right corner of the first report card (208A Ishii Haruko). A red line connects this icon to a 'Report Selection' modal window that is open over the grid. The modal window contains the following report options:

- Sleep Report
- Sleep Statistics Report
- Respiration Report
- Safety Report
- Safety Statistics Report
- Utilization Rate Report
- Dietary Report
- Dietary Statistics Report
- Dementia Report
- Dementia Statistics Report
- Indoor Environment
- Indoor Environment

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
67 27

1

# LES RAPPORTS DE SANTE

1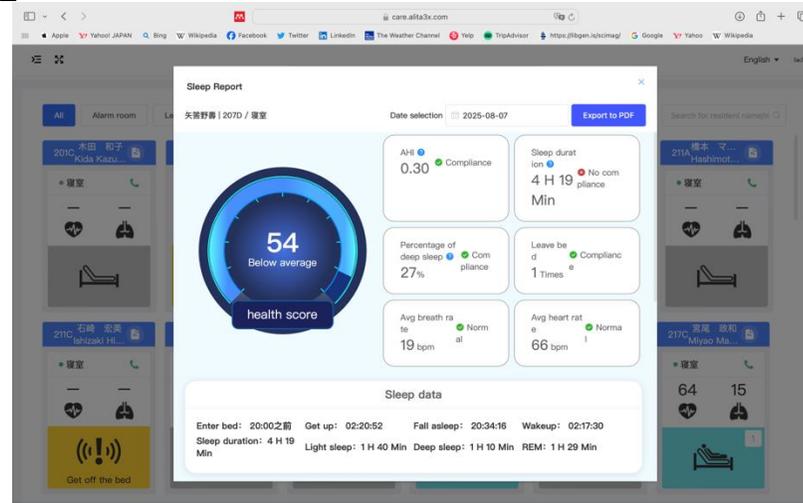


2

**Safety Report**

Date selection: 2025-08-07

Room/Type	Alarm	Time
202C/寢室	Bed exit alarm	2025-08-07 08:59:38
202C/寢室	Bed exit alarm	2025-08-07 05:06:37
202C/寢室	Sit up in bed	2025-08-07 05:04:59
202C/寢室	Bed exit alarm	2025-08-07 02:18:12
202C/寢室	Bed exit alarm	2025-08-07 02:09:20
202C/寢室	Bed exit alarm	2025-08-07 02:04:05
202C/寢室	Bed exit alarm	2025-08-07 01:44:18
202C/寢室	Bed exit alarm	2025-08-07 01:34:57
202C/寢室	Bed exit alarm	2025-08-07 01:17:14

3

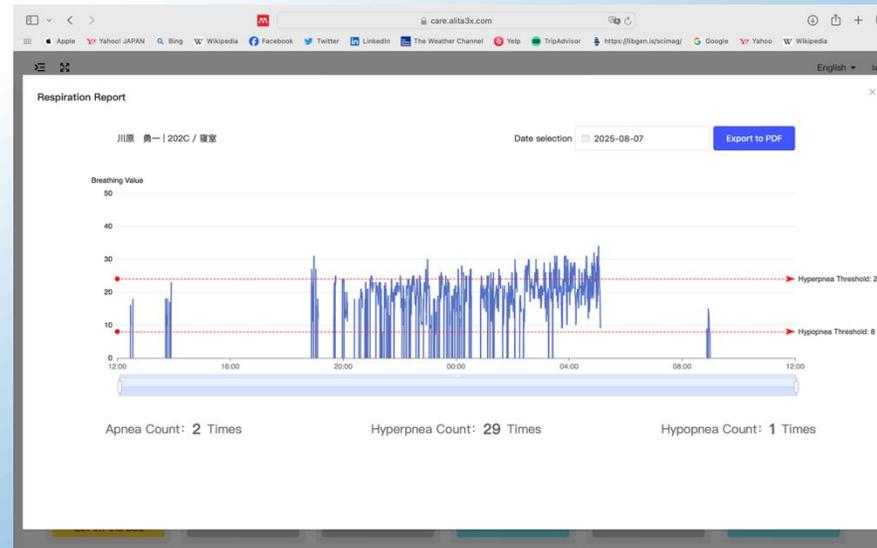**Report Selection**

1 Sleep Report    2 Sleep Statistics Report    3 Respiration Report

Safety Report    Safety Statistics Report

Utilization Rate Report

Dietary Report    Dietary Statistics Report

Dementia Report    Dementia Statistics Report

Indoor Environment    Indoor Environment

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